

DSSE

PRODUCT MANUAL



DECIMA TREATMENT BED TB001

Compact Powerhouse for Professional Treatments

This premium treatment bed is upholstered in durable, medical-grade vinyl and features 4 whisper-quiet motors for smooth adjustment of the backrest, legrest, height, and incline. Designed for versatility, the bed offers 210° stable rotation and can easily convert into a chair position—making it the perfect solution for smaller treatment rooms without compromising functionality or client comfort.

Introduction

This manual is intended solely as a reference for the operation of this specific model. DSSE assumes no responsibility or liability for any consequences resulting from its use for other purposes. All information contained in this manual is protected by copyright and may not be copied, reproduced, or translated, in whole or in part, without the prior written consent of our company.

Safety Instructions

- Always disconnect the power supply before changing electrical components or performing maintenance and cleaning on the equipment.
- The plug must be connected to a properly installed and grounded outlet to prevent the risk of fire or electric shock.
- Do not stand on the equipment.
- Keep children away from the equipment; place it out of their reach. Always adjust the backrest to its highest position when leaving the equipment unattended.
- Do not place the equipment in damp or wet areas such as bathrooms to avoid electric shock or malfunction. The remote control should also be kept away from water.
- Avoid using the equipment in environments where painting, coating, or oxygen provision is taking place.
- Unless supervised or instructed by qualified personnel, people with physical disabilities, sensory disorders, mental impairments, lack of experience or common sense, and children are not permitted to use the equipment.
- If the plug is damaged, contact your distributor, service center, or a qualified professional for repair.
- Ensure there are no objects within the equipment's movement range during operation.
- All maintenance must be performed by trained professionals only.

Precautions

- The equipment should be operated by experienced personnel familiar with its structure, performance, and instructions.
- Read this manual carefully before installation, operation, and adjustment. Keep it for future reference.
- Do not use for single-side or single-point loading.
- Follow safety operation rules. Do not overload. Ensure all safety equipment is reliable and address any hazards immediately.
- Regularly tighten bolts and nuts to avoid failure due to loosening or vibration.
- Do not disassemble parts without proper authorization.
- If faults occur or abnormal noises are heard, stop usage immediately and have the equipment inspected.
- Only resume use after proper troubleshooting.
- This bed uses a power cord and must be plugged into a grounded outlet.
- Observe all warning labels on the product.
- Clean dust and stains using a neutral cleaning agent after each treatment.
- Keep hands, feet, and head clear of moving parts during adjustments.
- Always observe the maximum user weight capacity of the bed.
- Do not sit or apply heavy force on armrests, head or back sections. Refer to the weight limits in this manual.
- Motor operation time: 2 minutes ON, 18 minutes OFF to prevent overheating).
- Clients should enter and exit the bed from the middle section only.

PRECISION ENGINEERING, STREAMLINED ELEGANCE

Decima Treatment Bed – Where Innovation Meets Comfort

Elevate your treatment experience with the Decima Treatment Bed, crafted for professionals who demand precision, durability, and style. Featuring smooth, motorized adjustments for the backrest, height, leg rest and incline via a smart foot controller, this bed offers seamless transitions with whisper-quiet performance. Upholstered in premium materials for both comfort and hygiene, it's the perfect fusion of ergonomic support and sleek design.

Adjustable (Pull-out) Headrest (18cm)

The pull-out headrest provides adjustable support for enhanced comfort, allowing you to extend or retract as needed to perfectly accommodate different clients and treatment positions.

Adjustable Backrest (0° - 80°)

Easily adjustable from 0° to 80°, the backrest allows you to customize the seating angle for optimal comfort and support.

Height Adjustment (66 - 96 cm)

Adjustable height for ergonomic comfort and ease of use.

Rotatable (±105°)

Smooth rotation up to 105° each way (from centre) for ultimate flexibility and ease.

Power Source

Power located beneath the head section at the rear of the base, featuring an integrated on/off switch and IEC socket for easy access and tidy cable management.



Removable & Swiveling Armrests (0°-180°)

The armrests swivel smoothly from 0° to 180° and can be removed as required, providing flexible positioning, comfortable support, and easy access during treatments or procedures.

Inclineable Seat (10°)

Enjoy personalized comfort with a seat that gently tilts up to 10°.

Adjustable Legrest (0°-80°)

Effortless adjustment for maximum comfort and support.

Pull-Out Footrest (13cm)

Easily extends to support legs, enhancing comfort during use.

Streamlined Base

The compact, curved base design allows effortless access during treatments.


Product Specifications

Length:	180cm
Width without/with Arms:	59/83cm
Height Range:	66-96cm
Backrest Adjustment Angle:	Up to 80°
Seat Tilt Adjustment Angle:	Up to 10°
Legrest Adjustment Angle:	Up to 80°
Net Weight:	90 kg
Gross Weight:	98 kg
Max. Load Capacity:	200 kg (seating position)
Power Supply:	AC 220-240V, 50Hz/60Hz
Max. Power Consumption:	300W



Slimline Foot Control Operation

The Decima Treatment Bed comes equipped with a slimline foot pedal as standard, and can be optioned with an intuitive hand-held remote for even greater control. Effortlessly adjust the height, backrest, tilt, and leg sections with smooth precision thanks to its responsive, soft-touch buttons designed for fast, fluid transitions. Whether you're repositioning mid-treatment or setting up between clients, the Decima ensures seamless operation, uninterrupted workflow, and superior ergonomic comfort for every professional.

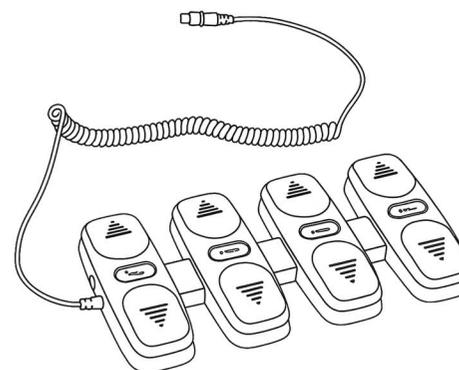

How To Use

Follow these instructions to operate the electric beauty bed safely and effectively.



Backrest Adjustment

Press to raise or lower the backrest



Legrest Adjustment

Press to raise or lower the legrest



Height Adjustment

Press to raise or lower the bed




Incline Adjustment

Press to tilt (incline) the bed

Installation Preparation: We recommend two (2) people for installation, as both individuals are required to lift and carefully remove the bed from the pallet onto the floor.


STEP 1

With the help of another person, carefully lift and remove the bed from the pallet and place it onto a flat, stable surface.

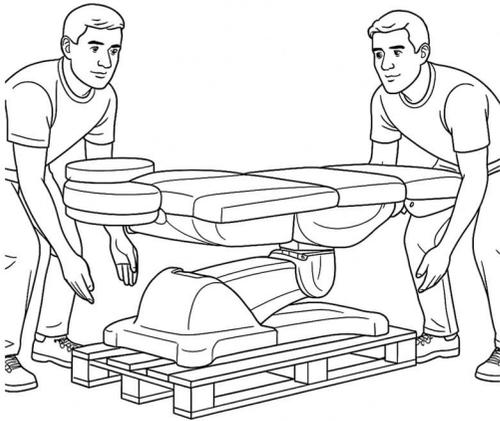



STEP 2

Remove the leg extension (B) from its packaging, loosen the knob under the legrest, insert the extension, and tighten the knob securely to lock it in place.

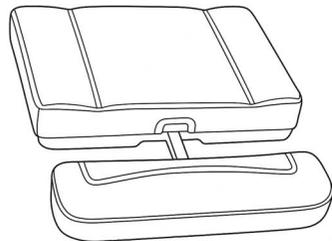

STEP 3

Remove the head rest (A) from its packaging, loosen the knob under the head rest, insert the head rest, and tighten the knob securely to lock it in place. Then insert the head pillow.

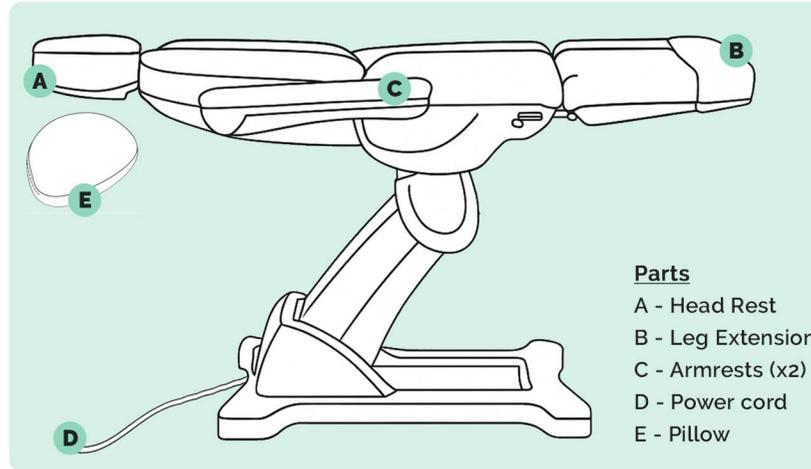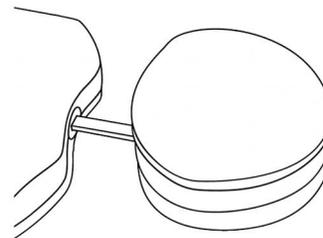


Parts

- A - Head Rest
- B - Leg Extension
- C - Armrests (x2)
- D - Power cord
- E - Pillow


STEP 4

Insert each armrest (C) by guiding the pole or bolt into the slot (D) on both sides of the chair. Ensure the armrests are facing upwards, as shown in the image below. Push firmly until you feel the armrests lock into place, confirming they are securely installed on each side.

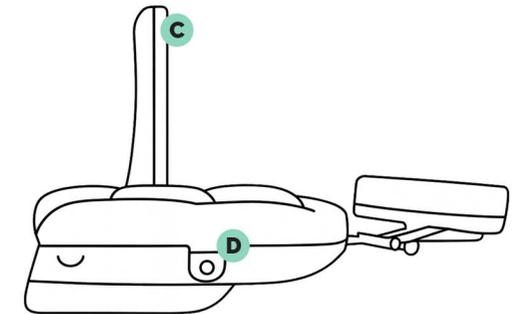

STEP 5

Plug the power cable into a standard wall socket, ensuring it is fully inserted. Make sure the power switch on the bed and the wall socket are both turned on before operating the bed.

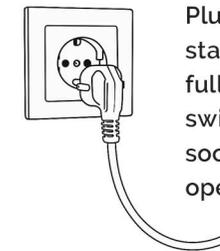

SAFETY WARNING: Do not allow children to operate this product. Always ensure that customers enter and exit the bed only from the middle section. Failure to follow these instructions may result in serious injury and damage to the bed.

Upholstery Care Guide

As a professional esthetician, investing in a high-quality electric facial bed can significantly enhance the comfort and effectiveness of your treatments. To maintain that level of performance and ensure long-term reliability, regular maintenance and proper care are essential. Like any professional equipment, your facial bed requires routine attention to stay in optimal working condition—ensuring safety, smooth functionality, and a consistently elevated experience for your clients.

General Care Tips

Maintain a stable temperature:

- Cold can lead to vinyl cracking
- Heat may cause permanent stretching

Keep out of direct sunlight for long periods to prevent fading and damage. Regular housekeeping helps preserve comfort and appearance.



Cleaning Instructions

Clean frequently to remove body oils that can harden vinyl

Use a colorfast cloth dampened with lukewarm water and mild, alkaline-free soap

Avoid soaking the material

Wipe off any soap with fresh water and a clean cloth

Dry thoroughly with a soft, dry cloth

For tougher grime, gently scrub with a soft brush



Avoid Using These Products

- Hot or boiling water
- Alcohol or hydrocarbon-based cleaners
- Acetone
- Bleach or harsh detergents
- Furniture polish
- Abrasive powders, pastes, or steel wool
- Silicone or petroleum-based products
- Solvent-based or abrasive cleaning agents

Routine Inspection

Perform periodic checks on:

- Hinges
- Fasteners
- Structural components

Ensure your chair remains safe, stable, and fully functional.

CE & RCM Certification

All electric components in our beauty beds are CE (Europe) & RCM (Australia & New Zealand) certified, meeting the strict safety, health, and environmental standards required in Australia, the EU, and the UK.



WARRANTY PERIOD

STRUCTURE: 5 YEARS | VINYL: 2 YEARS | MOTORS: 2 YEARS

For more information about our warranty, please refer to our website.



Australia:

www.dsse.com.au

United Kingdom:

www.dssebeauty.co.uk

Europe:

www.dsseworld.com