

DSSE

PRODUCT MANUAL



ELECTRIC BED 1 MOTOR BB11

Refined Simplicity with Everyday Reliability

Thoughtfully designed for beauty and wellness professionals, the Electric Bed 1 Motor combines essential functionality with dependable comfort. This 3-piece treatment couch features a smooth electric height adjustment via a single motor, while the backrest and leg sections are easily adjusted manually. Quiet, stable, and versatile, it's an ideal choice for clinics and salons seeking streamlined performance and ergonomic support in daily treatments.

Introduction

This manual is intended solely as a reference for the operation of this specific model. DSSE assumes no responsibility or liability for any consequences resulting from its use for other purposes. All information contained in this manual is protected by copyright and may not be copied, reproduced, or translated, in whole or in part, without the prior written consent of our company.

Safety Instructions

- Always disconnect the power supply before changing electrical components or performing maintenance and cleaning on the equipment.
- The plug must be connected to a properly installed and grounded outlet to prevent the risk of fire or electric shock.
- Do not stand on the equipment.
- Keep children away from the equipment; place it out of their reach. Always adjust the backrest to its highest position when leaving the equipment unattended.
- Do not place the equipment in damp or wet areas such as bathrooms to avoid electric shock or malfunction. The remote control should also be kept away from water.
- Avoid using the equipment in environments where painting, coating, or oxygen provision is taking place.
- Unless supervised or instructed by qualified personnel, people with physical disabilities, sensory disorders, mental impairments, lack of experience or common sense, and children are not permitted to use the equipment.
- If the plug is damaged, contact your distributor, service center, or a qualified professional for repair.
- Ensure there are no objects within the equipment's movement range during operation.
- All maintenance must be performed by trained professionals only.

Precautions

- The equipment should be operated by experienced personnel familiar with its structure, performance, and instructions.
- Read this manual carefully before installation, operation, and adjustment. Keep it for future reference.
- Do not use for single-side or single-point loading.
- Follow safety operation rules. Do not overload. Ensure all safety equipment is reliable and address any hazards immediately.
- Regularly tighten bolts and nuts to avoid failure due to loosening or vibration.
- Do not disassemble parts without proper authorization.
- If faults occur or abnormal noises are heard, stop usage immediately and have the equipment inspected.
- Only resume use after proper troubleshooting.
- This bed uses a power cord and must be plugged into a grounded outlet.
- Observe all warning labels on the product.
- Clean dust and stains using a neutral cleaning agent after each treatment.
- Keep hands, feet, and head clear of moving parts during adjustments.
- Always observe the maximum user weight capacity of the bed.
- Do not sit or apply heavy force on armrests, head or back sections. Refer to the weight limits in this manual.
- Motor operation time: 2 minutes ON, 18 minutes OFF to prevent overheating).
- Clients should enter and exit the bed from the middle section only.

BEAUTY ELEVATED BY DESIGN

Experience elevated comfort with the Electric Bed 1 Motor, a beautifully crafted treatment bed designed to deliver outstanding support across a wide range of beauty services. Ideal for brow and lash treatments, facials, massages, and more, its sleek base and plush upholstery add a modern, professional touch to any salon or spa.

Upholstered in premium medical-grade vinyl and layered with memory foam, this bed offers both durability and indulgent comfort. Equipped with a whisper-quiet motor for smooth height adjustment, it combines style, performance, and practicality, ensuring every treatment is as seamless as it is luxurious.

Adjustable Backrest (0° - 52°)

Easily adjustable via manual hand lever, the backrest can be positioned anywhere between 0° and 52°.

Height Adjustment (56 - 83 cm)

Adjustable height for ergonomic comfort and ease of use.

Elegance from the Ground Up

A slimline, sculpted base that lets you get closer to your client for precision and comfort in every treatment.



Wide Insert Pillow

Enjoy extra comfort with a wider pillow insert.

Ergonomic Comfort

Channeled curves contour to the body, offering superior comfort and support throughout treatments.

Adjustable Legrest (0°-42°)

Easily adjustable via manual hand lever, the backrest can be positioned anywhere between 0° and 42°.

Product Specifications

Length:	186cm
Width without/with Arms:	62/82cm
Height Range:	56-83cm
Backrest Adjustment Angle:	Up to 52°
Legrest Adjustment Angle:	Up to 42°
Net Weight:	70 kg
Gross Weight:	80 kg
Max. Load Capacity:	200 kg (seating position)
Power Supply:	AC 220-240V, 50Hz/60Hz
Max. Power Consumption:	300W




Effortless Hand Remote Control

The Electric Bed 1 Motor features an intuitive hand-held remote, with an optional foot control for hands-free height adjustment. Its soft-touch buttons provide smooth, effortless control, while a manual mechanism allows for easy adjustment of the backrest and leg sections. Designed to support ergonomic working conditions, this bed offers reliability and comfort for professionals delivering focused, high-quality care.

How To Use

Follow these instructions to operate the electric beauty bed safely and effectively.

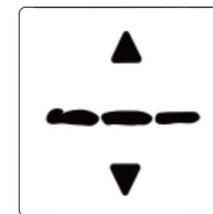

Height Adjustment


Press to raise or lower the bed

Leg and Backrest Adjustment

Both sections manually adjusted by hand lever




Installation Preparation: We recommend raising the bed to its maximum height using the hand control, then disconnect the power before continuing with the installation steps.

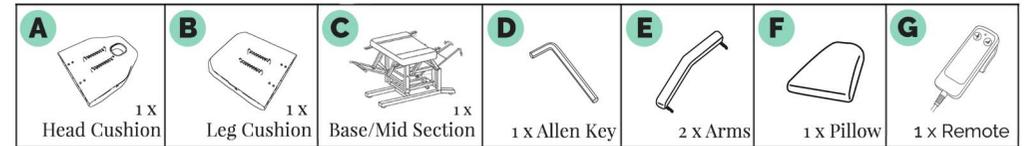


STEP 1

With two people, grasp the middle section (C) by holding the middle cushion. Carefully remove it from the pallet and place the base on the floor.

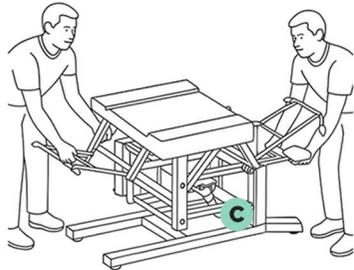

STEP 2

Locate the screws pre-fitted into the back of the head (A) and leg (B) cushions. Using the provided allen key (D), remove these screws.

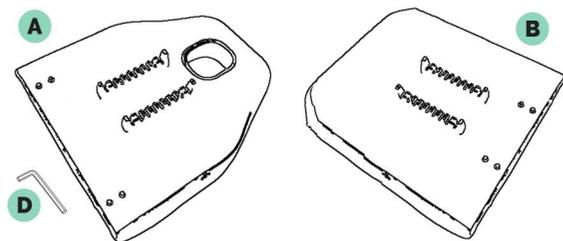


STEP 3

1. Line up the brackets (I) with the holes located in the head cushion (A).
2. Insert the hand lever (H) into the tracks located under the head cushion.
3. Carefully screw them in using the allen key provided.
4. Remove the safety pin and rod as shown in diagram (I).
5. Place the motor inside the bracket, then push the rod back through the bracket and motor.
6. Reinsert the safety pin and bend it securely into place.
7. Install leg cushion (B) by aligning brackets (I), inserting the hand lever into the tracks, and tightening four screws.




Warning: Take care when tightening screws to avoid cross-threading.

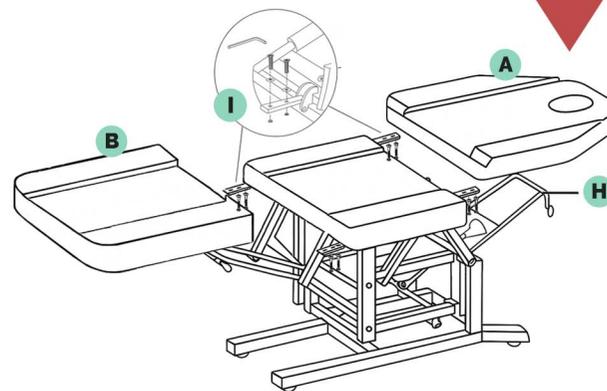


STEP 4

Insert the armrests (E) by gently bending them to align with the armrest holes in the sides of the bed top. Place the insert pillow (F) into the face hole.

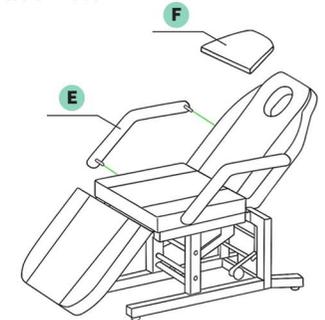

STEP 5

Reinsert the hand remote (G) into the control box and ensure it is fully connected. Plug the power cable into a standard wall socket. Once connected, the bed is ready for use.

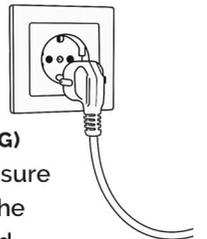

SAFETY WARNING: Turn off the power when the bed is not in use. Regularly inspect the power cord for any damage and keep it away from heat sources. Do not allow children to operate this product. Always ensure that customers enter and exit the bed only from the middle section. Failure to follow these instructions may result in serious injury and damage to the bed.

Upholstery Care Guide

As a professional esthetician, investing in a high-quality electric facial bed can significantly enhance the comfort and effectiveness of your treatments. To maintain that level of performance and ensure long-term reliability, regular maintenance and proper care are essential. Like any professional equipment, your facial bed requires routine attention to stay in optimal working condition—ensuring safety, smooth functionality, and a consistently elevated experience for your clients.

General Care Tips

Maintain a stable temperature:

- Cold can lead to vinyl cracking
- Heat may cause permanent stretching

Keep out of direct sunlight for long periods to prevent fading and damage. Regular housekeeping helps preserve comfort and appearance.



Cleaning Instructions

Clean frequently to remove body oils that can harden vinyl

Use a colorfast cloth dampened with lukewarm water and mild, alkaline-free soap

Avoid soaking the material

Wipe off any soap with fresh water and a clean cloth

Dry thoroughly with a soft, dry cloth

For tougher grime, gently scrub with a soft brush



Avoid Using These Products

- Hot or boiling water
- Alcohol or hydrocarbon-based cleaners
- Acetone
- Bleach or harsh detergents
- Furniture polish
- Abrasive powders, pastes, or steel wool
- Silicone or petroleum-based products
- Solvent-based or abrasive cleaning agents

Routine Inspection

Perform periodic checks on:

- Hinges
- Fasteners
- Structural components

Ensure your chair remains safe, stable, and fully functional.

CE & RCM Certification

All electric components in our beauty beds are CE (Europe) & RCM (Australia & New Zealand) certified, meeting the strict safety, health, and environmental standards required in Australia, the EU, and the UK.



WARRANTY PERIOD

STRUCTURE: **5 YEARS** | VINYL: **2 YEARS** | MOTOR: **2 YEARS**

For more information about our warranty, please refer to our website.



Australia:

www.dsse.com.au

United Kingdom:

www.dssebeauty.co.uk

Europe:

www.dsseworld.com