

DSSE

PRODUCT MANUAL



OCTAVIA TREATMENT BED TB002

The Epitome of Luxury and Performance for Skin Professionals

Indulge your clients in uncompromised comfort with this ultra-luxurious electric beauty bed, meticulously crafted for discerning skin professionals. Upholstered in premium materials and generously cushioned with plush memory foam, it offers a sanctuary of relaxation. Its sleek, modern silhouette is paired with seamless motorized adjustability, allowing effortless transitions between treatments. Designed to elevate both client experience and practitioner ease, this bed is the ultimate statement of sophistication in any high-end salon or clinic.

Introduction

This manual is intended solely as a reference for the operation of this specific model. DSSE assumes no responsibility or liability for any consequences resulting from its use for other purposes. All information contained in this manual is protected by copyright and may not be copied, reproduced, or translated, in whole or in part, without the prior written consent of our company.

Safety Instructions

- Always disconnect the power supply before changing electrical components or performing maintenance and cleaning on the equipment.
- The plug must be connected to a properly installed and grounded outlet to prevent the risk of fire or electric shock.
- Do not stand on the equipment.
- Keep children away from the equipment; place it out of their reach. Always adjust the backrest to its highest position when leaving the equipment unattended.
- Do not place the equipment in damp or wet areas such as bathrooms to avoid electric shock or malfunction. The remote control should also be kept away from water.
- Avoid using the equipment in environments where painting, coating, or oxygen provision is taking place.
- Unless supervised or instructed by qualified personnel, people with physical disabilities, sensory disorders, mental impairments, lack of experience or common sense, and children are not permitted to use the equipment.
- If the plug is damaged, contact your distributor, service center, or a qualified professional for repair.
- Ensure there are no objects within the equipment's movement range during operation.
- All maintenance must be performed by trained professionals only.

Precautions

- The equipment should be operated by experienced personnel familiar with its structure, performance, and instructions.
- Read this manual carefully before installation, operation, and adjustment. Keep it for future reference.
- Do not use for single-side or single-point loading.
- Follow safety operation rules. Do not overload. Ensure all safety equipment is reliable and address any hazards immediately.
- Regularly tighten bolts and nuts to avoid failure due to loosening or vibration.
- Do not disassemble parts without proper authorization.
- If faults occur or abnormal noises are heard, stop usage immediately and have the equipment inspected.
- Only resume use after proper troubleshooting.
- This bed uses a power cord and must be plugged into a grounded outlet.
- Observe all warning labels on the product.
- Clean dust and stains using a neutral cleaning agent after each treatment.
- Keep hands, feet, and head clear of moving parts during adjustments.
- Always observe the maximum user weight capacity of the bed.
- Do not sit or apply heavy force on armrests, head or back sections. Refer to the weight limits in this manual.
- Motor operation time: 2 minutes ON, 18 minutes OFF to prevent overheating).
- Clients should enter and exit the bed from the middle section only.

REFINED COMFORT, EXCEPTIONAL AESTHETICS

The Octavia Treatment Bed is a highly versatile and elegantly engineered solution for cosmetic, facial, body, and advanced skin treatments. Upholstered in durable anti-microbial medical-grade vinyl, it promotes a hygienic, professional environment while delivering exceptional comfort for daily use.

Designed with flexibility in mind, the Octavia features removable armrests, an adjustable headrest, and a legrest extension, allowing effortless adaptation to a wide range of treatments and client requirements. Three quiet electric motors provide smooth electronic adjustment of the height, backrest, and leg sections, operated via a convenient handheld remote with an optional foot control for hands-free positioning. With its sleek silhouette and intelligent functionality, the Octavia is a refined and practical centrepiece for any modern treatment space.

Adjustable Backrest (0° - 80°)

Easily adjustable from 0° to 80°, the backrest allows you to customize the seating angle for optimal comfort and support.

Height Adjustment (68 - 88 cm)

Adjustable height for ergonomic comfort and ease of use.

Adjustable Legrest (0°-80°)

Effortless adjustment for maximum comfort and support.

Power Source

Power board located beneath the head section at the rear, featuring an integrated on/off switch and IEC socket for easy access and tidy cable management.

Adjustable (Pull-out) headrest (18cm)

The pull-out headrest provides adjustable support for enhanced comfort, allowing you to extend or retract as needed to perfectly accommodate different clients and treatment positions.

Removable & Swiveling Armrests (0°-180°)

The armrests swivel smoothly from 0° to 180° and can be removed as required, providing flexible positioning, comfortable support, and easy access during treatments or procedures.

Luxurious Ergonomic Comfort

The channeled stitching not only adds a refined aesthetic but delivers superior, ultra-plush comfort—cradling the body with precision support for a truly indulgent treatment experience.

Pull-Out Footrest (13cm)

Easily extends to support legs, enhancing comfort during use.


Sleek, Clinical Elegance

The slimline base and electric column deliver a refined, modern aesthetic—ideal for today's professional treatment environments.



Product Specifications

Length:	193cm
Width without/with Arms:	60/82cm
Height Range:	68-88cm
Backrest Adjustment Angle:	Up to 80°
Legrest Adjustment Angle:	Up to 80°
Net Weight:	85 kg
Gross Weight:	94 kg
Max. Load Capacity:	200 kg (seating position)
Power Supply:	AC 220-240V, 50Hz/60Hz
Max. Power Consumption:	300W

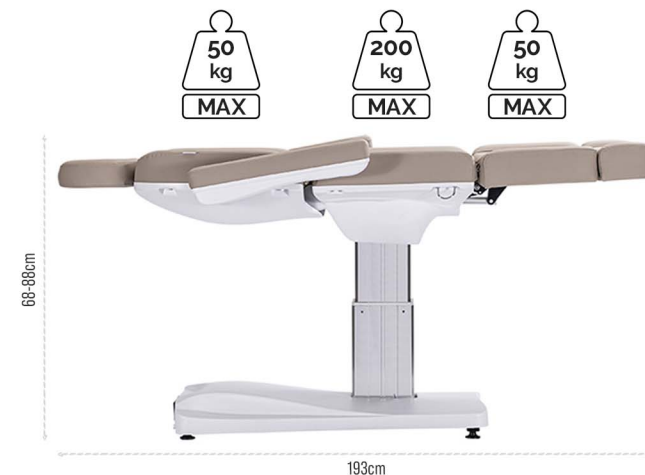

Effortless Hand Remote Control

The Octavia treatment bed is equipped with an intuitive hand-held remote (optional foot remote available) that allows effortless adjustment of height, backrest, and leg sections. Its responsive soft-touch buttons provide smooth, precise positioning to suit your working needs—ensuring uninterrupted focus during treatments and enhanced ergonomic comfort for every professional.

How To Use

Follow these instructions to operate the electric beauty bed safely and effectively.



Backrest Adjustment

Press to raise or lower the backrest



Legrest Adjustment

Press to raise or lower the legrest




Height Adjustment

Press to raise or lower the bed

Installation Preparation: We recommend two (2) people for installation, as both individuals are required to lift and carefully remove the bed from the pallet onto the floor.


STEP 1

With the help of another person, carefully lift and remove the bed from the pallet and place it onto a flat, stable surface..

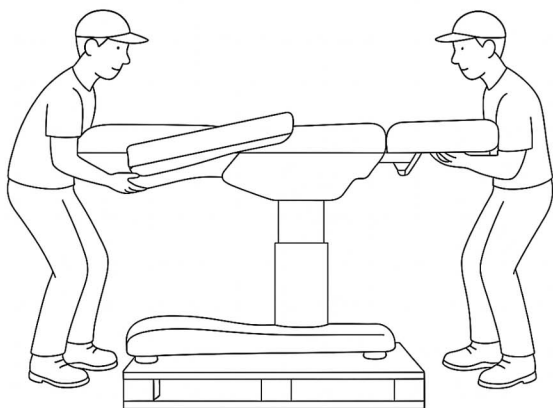



STEP 2

Remove the leg extension (B) from its packaging and insert it as shown.

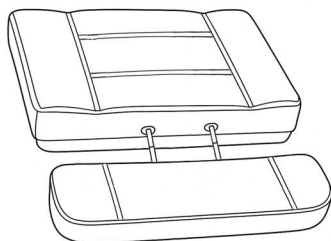


STEP 3

Remove the head rest (A) from its packaging and insert the head rest as shown below. Then insert the head pillow.

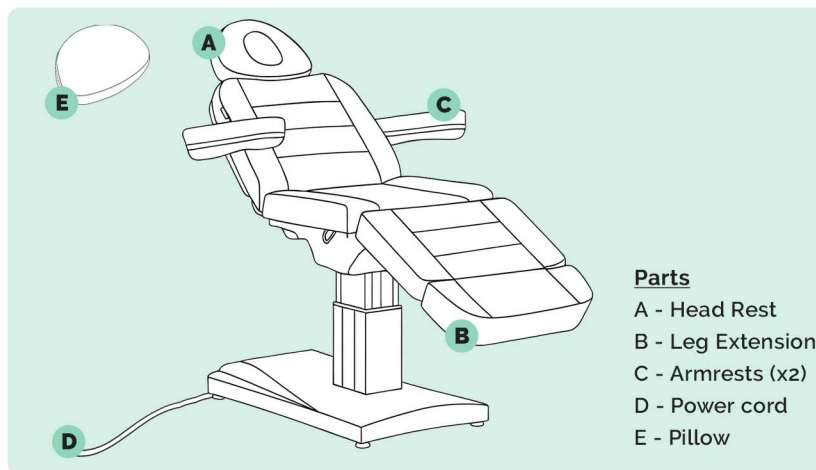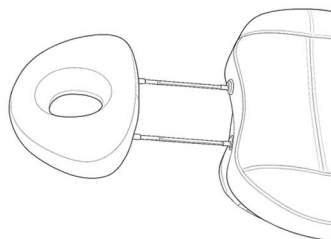


STEP 4

Insert each armrest (C) by guiding the pole or bolt into the slot (D) on both sides of the chair. Ensure the armrests are facing upwards, as shown in the image below. Push firmly until you feel the armrests lock into place, confirming they are securely installed on each side.

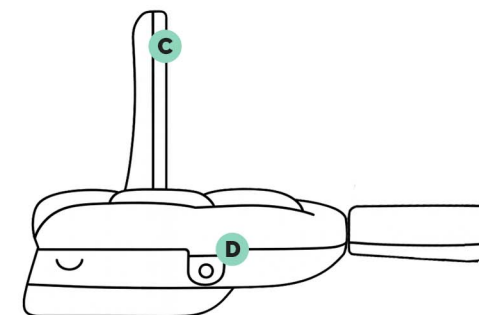

STEP 5

Plug the power cable into a standard wall socket, ensuring it is fully inserted. Make sure the power switch on the bed and the wall socket are both turned on before operating the bed.

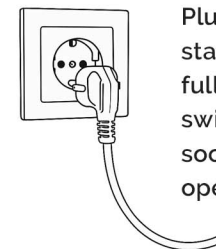

SAFETY WARNING: Do not allow children to operate this product. Always ensure that customers enter and exit the bed only from the middle section. Failure to follow these instructions may result in serious injury and damage to the bed.

Upholstery Care Guide

As a professional esthetician, investing in a high-quality electric facial bed can significantly enhance the comfort and effectiveness of your treatments. To maintain that level of performance and ensure long-term reliability, regular maintenance and proper care are essential. Like any professional equipment, your facial bed requires routine attention to stay in optimal working condition—ensuring safety, smooth functionality, and a consistently elevated experience for your clients.

General Care Tips

Maintain a stable temperature:

- Cold can lead to vinyl cracking
- Heat may cause permanent stretching

Keep out of direct sunlight for long periods to prevent fading and damage. Regular housekeeping helps preserve comfort and appearance.



Cleaning Instructions

Clean frequently to remove body oils that can harden vinyl

Use a colorfast cloth dampened with lukewarm water and mild, alkaline-free soap

Avoid soaking the material

Wipe off any soap with fresh water and a clean cloth

Dry thoroughly with a soft, dry cloth

For tougher grime, gently scrub with a soft brush



Avoid Using These Products

- Hot or boiling water
- Alcohol or hydrocarbon-based cleaners
- Acetone
- Bleach or harsh detergents
- Furniture polish
- Abrasive powders, pastes, or steel wool
- Silicone or petroleum-based products
- Solvent-based or abrasive cleaning agents

Routine Inspection

Perform periodic checks on:

- Hinges • Fasteners • Structural components

Ensure your chair remains safe, stable, and fully functional.

CE & RCM Certification

All electric components in our beauty beds are CE (Europe) & RCM (Australia & New Zealand) certified, meeting the strict safety, health, and environmental standards required in Australia, the EU, and the UK.



WARRANTY PERIOD

STRUCTURE: 5 YEARS | VINYL: 2 YEARS | MOTORS: 2 YEARS

For more information about our warranty, please refer to our website.

DSSE

Australia:

www.dsse.com.au

United Kingdom:

www.dssebeauty.co.uk

Europe:

www.dsseworld.com

Do not dispose of old or defective appliances with household waste. Please use designated public collection points for proper disposal.

This manual is for reference only. Specifications may change without prior notice.